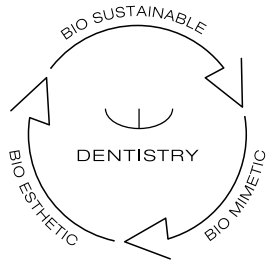
INDICATION GUIDE

DISCOVER YOUR SMILE

edelweiss
DENTISTRY

shaping the future of dentistry

AT ONE GLANCE...

Stephan Lamp
CEO & Founder of
edelweiss dentistry

Claudio Novelli
Clinical and
Scientific Director

edelweiss dentistry stands for the alliance between inspiration and technical know-how. VENEERs from edelweiss is state of the art for modern and minimally invasive esthetic dentistry. For the first time in the history of dental, it is now possible to work with prefabricated veneers made from nano-hybrid composite using modern laser technology. Never before has it been feasible to directly create the natural shape and youthful luminance of a tooth, so easily and perfectly in only one appointment. Its versatile area of application together with its time and cost saving procedure make edelweiss VENEERs a sound investment in the future with the best interest of the patient in mind. Convince yourself!

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DIRECT SYSTEM

THE PROGRESSIVE TECHNICAL CONCEPT

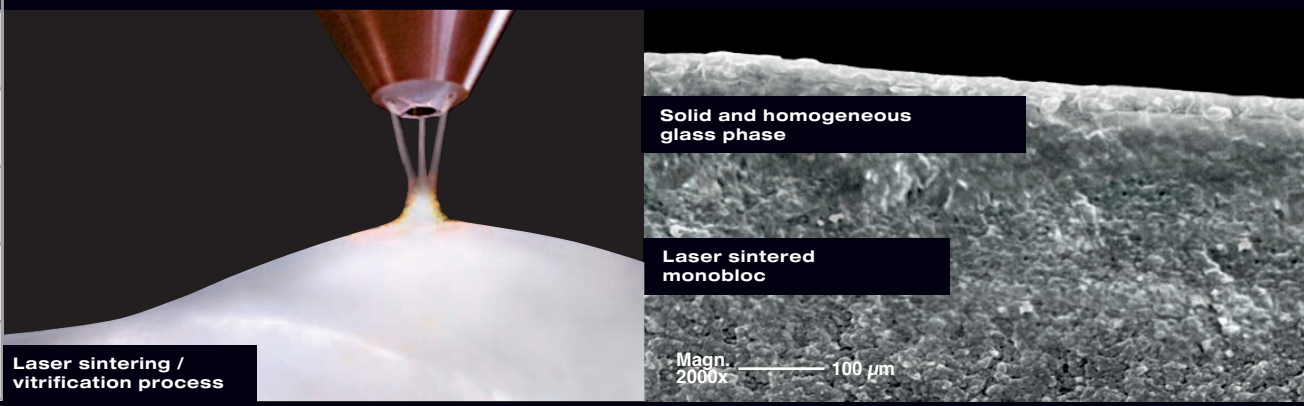
By combining and refining the latest composite & laser technology and achieving the optimum surface, edelweiss is the natural contemporary restorative option.

The difference is in it's similarity to nature.

| edelweiss | VENEER |
|----------------------|---------------|
| Flexural Strength | 200 MPa |
| Compressive Strength | 550 MPa |
| Flexural Modulus | 20 GPa |
| Surface Hardness | 100 HV |
| Filler Ratio | 82 % |

(Source: internal data from edelweiss dentistry)

Laser sintering / vitrification process

Solid and homogeneous glass phase

Laser sintered monobloc

Magn. 2000x 100 µm

SHADES

edelweiss SHADE SYSTEM

ENAMEL VENEER & OCCLUSIONVD + DENTIN / ENAMEL

edelweiss VENEERS & OCCLUSIONVD consist of the shade Enamel - Vita Enamel A0. The respective dentin and enamel shades used to cement the edelweiss VENEER & OCCLUSIONVD will determine the final shade tone of the restoration.

Example: edelweiss VENEER (Vita Enamel Shade A0) cemented with Dentin Shade A3 (Vita Dentin Shade A3) will result in the Vita Dentin Shade A3.

ENAMEL

Shade of edelweiss VENEER & OCCLUSIONVD

| | | | | | | |
|---------------|-------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------|---------------------------------------|----------------------------------|
| | | | | | | |
| Dentin | VENEER & OVD + Dentin A0 | VENEER & OVD + Dentin A1 | VENEER & OVD + Dentin A2 | VENEER & OVD + Dentin A3 | VENEER & OVD + Dentin A3.5 | VENEER & OVD + Enamel |
| VITA | A0 | A1 | A2 | A3 | A3.5 | Enamel |

FOR BRIGHTNESS & MAXIMUM BLENDING IN OF NATURAL TOOTH COLOR AND LUMINANCE

Cervical area 0.2 mm

Facial area 0.6 mm

Incisal edge 1.0 - 1.3 mm

MAXILLA

MANDIBULA

Marginal area: 0.2 mm

Occlusion area: 0.6 - 1.3 mm

DIRECT SYSTEM VENEER OCCLUSIONVD

INDICATIONS

Anterior and posterior restorations
Tooth discolorations
Anatomical deformities
Diastema
Erosion & Attrition
Customized Veneers, Crowns & Bridges
Crowns & Crown facings
Lifting the vertical dimension (CMD)

ADVANTAGES

One appointment
Minimally invasive
Natural look
Long-lasting
Biomechanical
Biocompatible
Antibacterial
Economical

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MISALIGNED TEETH / OLD RESTORATIONS

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FR: Malformations anatomiques

IT: Deformità anatomiche

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ANATOMICAL DEFORMITIES / LENGTHENING OF THE INCISAL EDGE

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