

edelweiss CAD/CAM BLOCKS

Step by Step

TOOTH PREPARATION



1
Pre-op, non-prep.



2
Try in of veneer.



3
Etching with 37 % phosphoric acid.



4
Rinse thoroughly with water to remove etchant.



5
Application of adhesive bonding agent to tooth surface. Gently air dry.



6
Light-cure for 20 seconds.



7
Placing a thin layer of edelweiss Bond on the veneer and light-cure for 10 seconds.



8
Evenly adapt the Nano-Hybrid Composite within the veneer using an instrument (spatula).



9
Placing veneer.



10
Removal of excess material at the marginal area.



11
Light-cure for 20 seconds.



12
Finishing



13
Retraction cord removal.



14
Finishing strip



15
Post-op