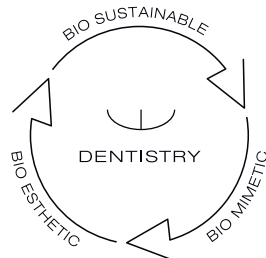
INDICATION GUIDE

DISCOVER YOUR SMILE

edelveiss
DENTISTRY

shaping the future of dentistry

AT ONE GLANCE ...

The **edelweiss SYSTEMS** with its natural concept of the layering technique is the quintessence of bio-esthetics, bio-mimetics and bio-compatible restorations in direct, semi-direct and indirect digital workflows.

It offers the clinical user as well as the technician a precise understanding of the inner structure of the natural tooth, its optical as well as functional properties and enables the development of unsurpassed bio-esthetics and bio-function with simple, controllable technical and clinical procedures.

Its unique patented production process consists of a modern laser sintering and vitrification process that produces a material with outstanding physical and esthetic properties.

The end result of this manufacturing process is a single glass-ceramic phase embedded in a carbon matrix to obtain a material that:

1. resembles esthetics of pure glass ceramics
2. strength of particle filled ceramics
3. simulates the modulus of elasticity of dentin

The bio-mechanical properties have been reconstructed optimally, merged together with natural bio-esthetics.

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DIRECT SYSTEM

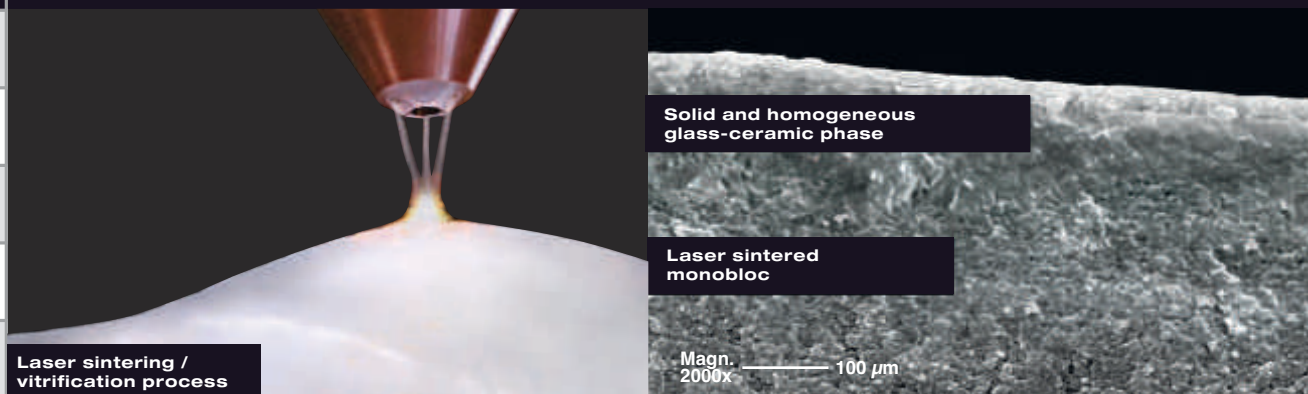
THE PROGRESSIVE TECHNICAL CONCEPT

By refining the latest laser technology and achieving the optimum surface, edelweiss is the natural contemporary restorative option.

The difference is in it's similarity to nature.

edelweiss	VENEER
Flexural Strength	200 MPa
Compressive Strength	550 MPa
Flexural Modulus	20 GPa
Surface Hardness	100 HV
Filler Ratio	82 %

(Source: internal data from edelweiss dentistry)

SHADES

edelweiss SHADE SYSTEM

ENAMEL **VENEER & OCCLUSIONVD** + **DENTIN** / **ENAMEL**

edelweiss VENEERS & OCCLUSIONVD consist of the shade Enamel – Vita Enamel AO. The respective dentin and enamel shades used to cement the edelweiss VENEER & OCCLUSIONVD will determine the final shade tone of the restoration.

Example: edelweiss VENEER (Vita Enamel Shade AO) cemented with Dentin Shade A3 (Vita Dentin Shade A3) will result in the Vita Dentin Shade A3.

ENAMEL

Shade of edelweiss
VENEER & OCCLUSIONVD

VENEER & OVD
+ Enamel

FOR BRIGHTNESS & MAXIMUM BLENDING IN
OF NATURAL TOOTH COLOR AND LUMINANCE

Cervical area 0.2 mm

Facial area 0.6 mm

Incisal edge 1.0 - 1.3 mm

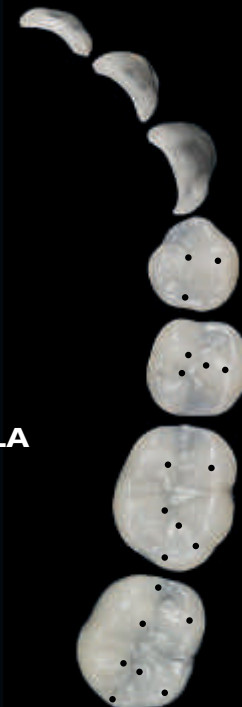
Marginal area: 0.2 mm

Occlusion area: 0.6 - 1.3 mm

MAXILLA

MANDIBULA

DIRECT SYSTEM

VENEER OCCLUSIONVD

INDICATIONS

Anterior and posterior restorations
Tooth discolorations
Anatomical deformities
Diastema
Erosion & Attrition
Customized Veneers, Crowns & Bridges
Crowns & Crown facings
Lifting the vertical dimension (CMD)

ADVANTAGES

One appointment
Minimally invasive
Natural look
Long-lasting
Biomechanical
Biocompatible
Antibacterial
Economical

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MISALIGNED TEETH / OLD RESTORATIONS

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FR: Malformations anatomiques

IT: Deformità anatomiche

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ANATOMICAL DEFORMITIES / LENGTHENING OF THE INCISAL EDGE

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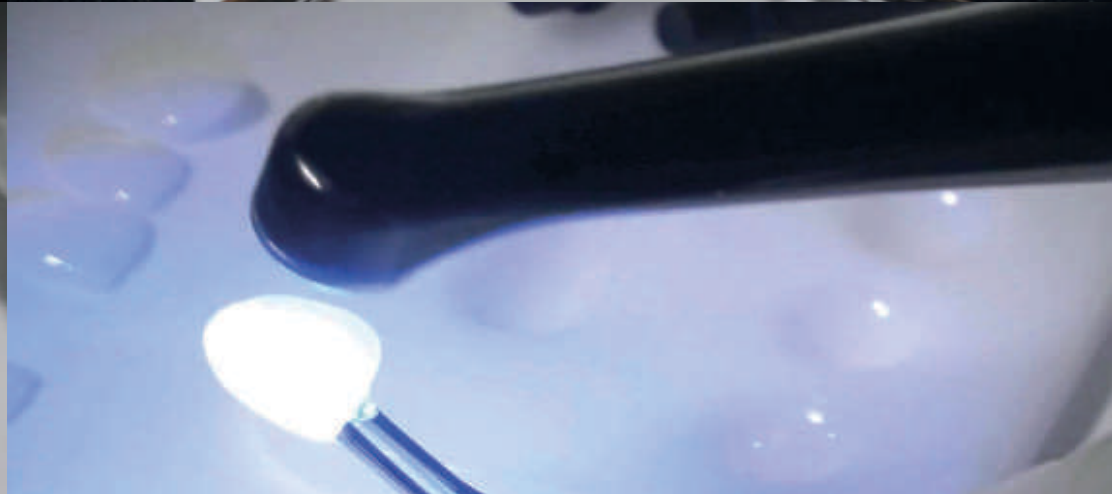
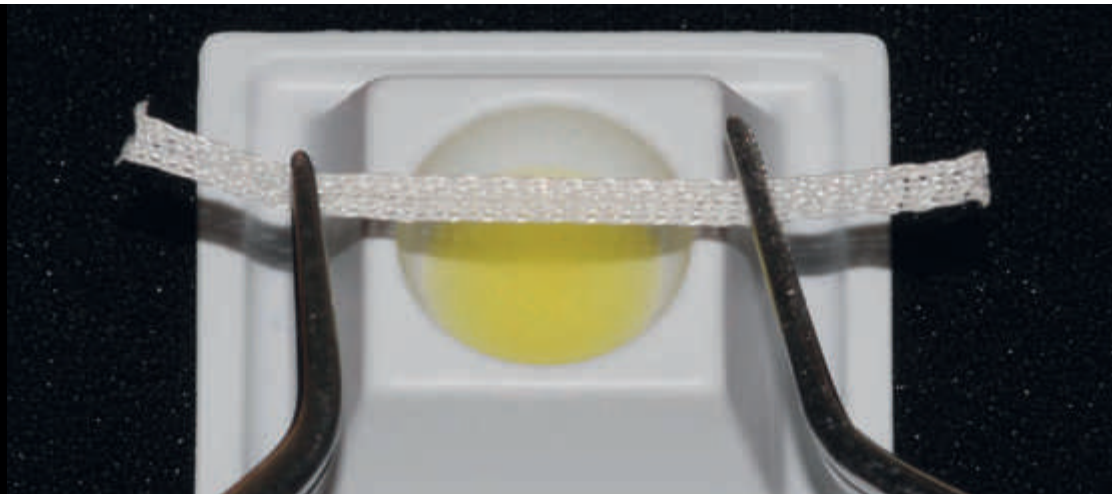
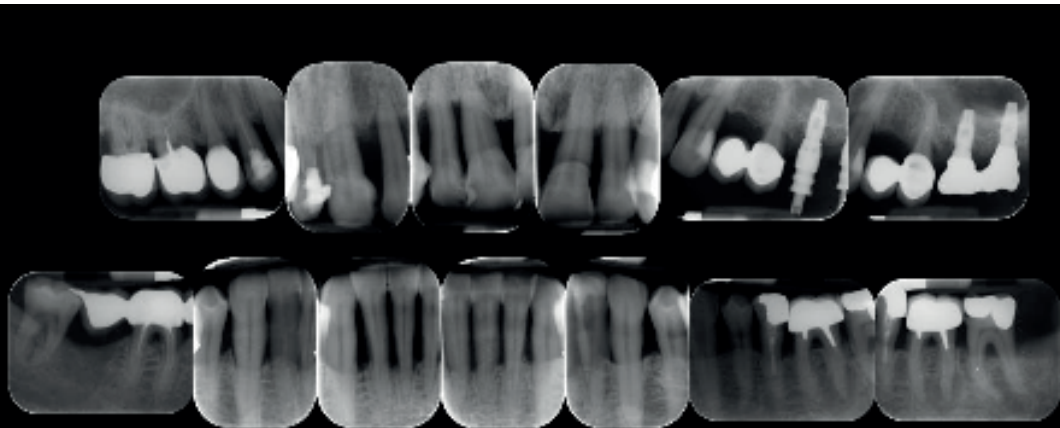
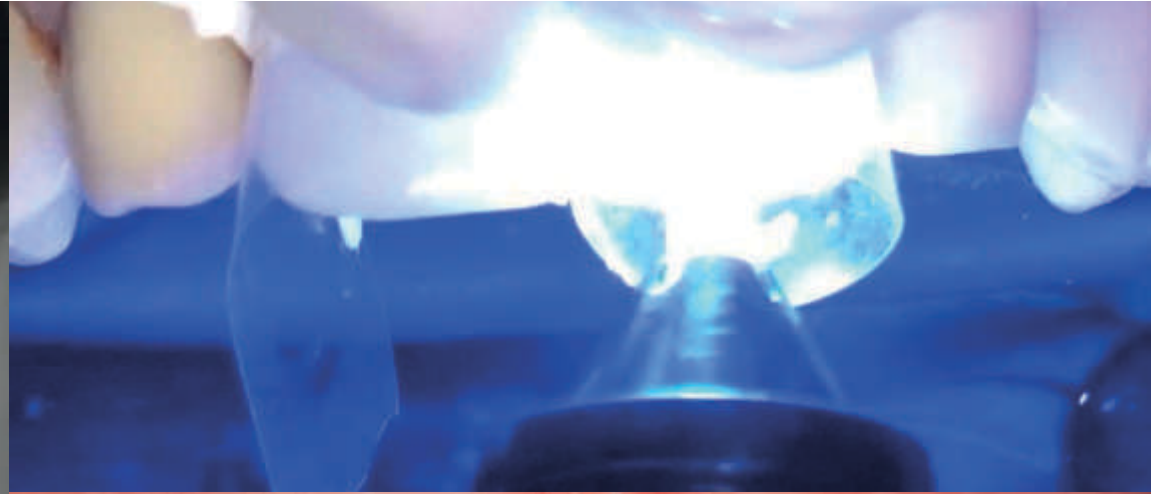
shaping the future of dentistry

edelweiss dentistry ® and the edelweiss Logo ® are registered trademarks of edelweiss dentistry products gmbh • Austria

© 2026 edelweiss dentistry products gmbh • Austria. All material contained in this brochure are the property of edelweiss dentistry products gmbh • Austria. No part of this publication may be reproduced, distributed, or transmitted in any form or by any means, without the prior written permission of the edelweiss dentistry products gmbh • Austria. All Rights Reserved.

edelweiss dentistry products gmbh • Austria
office@edelweissdentistry.com
www.edelweissdentistry.com

816005 / 2026-07